

**SUPER SHAKER BED VIBRATOR MODEL SS12V**

**IMPORTANT**– Please read these instructions carefully before use and retain for future reference. See operating instructions.

-- Use only an approved Sonic Alert extension cord model SBE115, or connect directly to unit.

- Warning:** To prevent fire or shock hazard do not expose this appliance to rain or moisture.
- No naked flame sources, such as lighted candles should be placed on the apparatus.
  - It shall not be exposed to dripping or splashing and that no objects filled with liquids, such as vases, shall be placed on the apparatus.
  - Do not expose the unit to any sources of water or moisture, do not place any objects with water or filled with liquids of any type near the unit.

**OPERATION**

When activated by your **Sonic Boom™ Clock** or any clock with matching DC outlet. **Sonic Alert Super Shaker** bed vibrator will wake even the heaviest sleeper.

The **Super Shaker Model SS12V** bed vibrator has extra strong vibrations designed especially for a long life. The **Super Shaker** bed vibrator also has a built-in automatic shut-off sensor so it will not overheat even if accidentally left on.

**INSTALLATION**

The **Super Shaker** bed vibrator is very safe and easy to use.

1. Place the **Super Shaker** bed vibrator between your mattress and box springs or under your pillow.
2. Plug the Super Shaker bed vibrator into the **Sonic Boom™ Clock** or into any clock with a matching DC outlet.
3. The **Super Shaker** bed vibrator will vibrate until you turn off the alarm or hit the snooze button. The Super Shaker bed vibrator will not overheat if you forget to turn off the alarm. If the Super Shaker bed vibrator is left on for more than approximately 10 minutes, the built-in temperature sensor will turn off the vibrator so it does not become too hot. When the Super Shaker bed vibrator cools down, the temperature sensor will automatically reset the vibrator for normal operation.

**SONIC ALERT'S LIMITED WARRANTY**

The SUPER SHAKER bed vibrator, Model SS12V is warranted against manufacturing defects in materials and workmanship for five (5) years from the date of purchase. Within this period Sonic Alert will repair or replace the SUPER SHAKER bed vibrator without charge for parts or labor. Simply send the SUPER SHAKER bed vibrator (postpaid) and your sales slip as proof of purchase to:

**Parc de l'étoile, 2 rue Galilée BP20061,  
59791 Grande Synthe Cedex**

 This icon indicates separate disposal for electrical and electronic equipment.

The product is covered by the legal guarantee of conformity as provided by applicable law.

**Recycling & the WEEE directive**

**At the end of their useful life, the packaging, product and batteries should be disposed of via a suitable recycling centre. Do not dispose of with your normal household waste. Do not burn.**

**Recycling**

All plastic and metal parts are recyclable. The printed circuit board may be sent to any PCB recovery contractor to recover some of the components for any metals such as gold and silver.

Local recycling: your local library or district council will also have information about recycling in your area.

**WEEE stands for Waste Electrical and Electronic Equipment**

The Directive (2012/19/EU) aims to reduce the waste arising from electrical and electronic equipment; and improve the environmental performance of all those involved in the life cycle of electrical and electronic products.



**Safety**

This product complied with EC Council Directive on electromagnetic compatibility 2004/108/EC with applicable standard(s) with amendments:  
EN61000-6-3:2007 +A1:2011  
EN61000-6-1:2007  
EN61000-3-2:2006 +A1:2009 +A2:2009  
EN61000-3-3:2008

This product complied with EC Low Voltage Directive 2006/95/EC with applicable standard(s) with amendments:  
EN60065:2002 +A1:2006 +A11:2008 +A2:2010 +A12:2011

Distributed by :



Parc de l'étoile, 2 rue Galilée BP20061,  
59791 Grande Synthe Cedex  
Tel: +33 (0) 3 28 58 75 75  
Fax: +33 (0) 3 28 58 75 76  
[www.geemarc.com](http://www.geemarc.com)

