FRONT

English

MODEL:RHK009 093-BC009-052

ALARM CLOCK WITH BED SHAKER
MODEL SBC575ss

OPERATION INSTRUCTIONS

IMPORTANT - Please read these instructions carefully before use and retain for future reference. See operating instructions on back.

- Use only the supplied or recommended Sonic Alert adapter to connect the unit.

Warning: To prevent fire or shock hazard do not expose this appliance to rain or moisture. Before connecting to mains power make sure that the requirements specified on this apparatus corresponds to the available power supply in that country.
I.e. AC100-240V~, 50/60 Hz.

- The ventilation should not be impeded by covering the ventilation openings with items such as, newspaper, table cloth, curtains, etc;

- No naked flame sources, such as lighted candles should be placed on the apparatus;

- It shall not be exposed to dripping or splashing and that no objects filled with liquids, such as vases, shall be placed on the apparatus;

- Do not place the product in direct bookcases or racks without proper ventilation.

- Where the mains' power plug is used as the disconnect device, the disconnect device shall remain readily operable.

GENERAL DATA

Powered Adapter: AC100-240V~, 50/60Hz
Power Consumption: Adapter Output: DC5V=~, 650mA

Test Mode:

1) Press and hold the Snooze and Time buttons for 3 seconds to enter the Test Mode. The display will show "TEST" to indicate it is in TEST Mode.
2) If within 1 minute no button is pressed, then it will exit the Test Mode and return back to the Main Mode.

3) Press any button within 1 minute (except the Snooze & Time buttons, which has no effect), it will exit the Test Mode immediately.

4) While in Test Mode, sliding the Function Switch will demonstrate different functions of the device:
a) Function Switch at OFF, it will have a continuous "Vee" sound to indicate it is in the test mode;
b) Function Switch at BUZZ, it will demonstrate the buzzer sound;
c) Function Switch at VIB, it will demonstrate the vibrator;
d) Function Switch at VIB & BUZZ, it will demonstrate the buzzer sound & the vibrator.

5) After finish with the Test Mode, press any button (except the Snooze & Time buttons, which has no effect) it will exit the Test Mode immediately.

Setting the Correct Time

1) Press and hold the TIME Button (6) and at the same time press the FAST Button (9) or the SLOW button (10) to advance clock display until you have desired time.
2) The lighted (2) located in upper left corner of the clock display indicates the time is PM. For AM, this date will not light up.

Setting the Alarm

1) Slide the ALARM MODE SWITCH (12) to AL1 position, the alarm indicator AL1 (1) will light up.
2) Press and hold the ALARM SET (5) button and at the same time press the FAST Button (9) or the SLOW button (10) to advance clock display until you reach desired alarm setting.
3) Make sure alarm time is correctly set for AM or PM (1).

Selecting 12 or 24 hours format

1) Once alarm has been set, slide ALARM MODE SWITCH (12) to AL2 position, the alarm indicator AL2 (17) will light up.
2) Press and hold the ALARM SET (5) button and at the same time press the FAST Button (9) or the SLOW button (10) to advance clock display until you reach desired alarm setting.

NOTE: The Alarm function of Alarm 1 and Alarm 2 is independent of each other.

Dimmer Operation

There is a 5-level brightness control for the LED display. Press the SNOOZE button once to change display brightness. The brightness will cycle from 5-4-3-2-1-2-3-4-5 (from bright to dim then from dim to bright).
Press and release the hour button to toggle between 12 or 24 hours. If no selection is made within 4 seconds, then the last displayed hour format will be chosen and the display will return to the normal time mode.

Volume Control

1) To adjust the volume, rotate VOLUME wheel (4) located on the lower right corner of the clock.

Tone Control

1) To adjust the TONE, turn the TONE wheel (14) located on the top right corner of the clock.

Bed Shaker / Vibrator

1) Attach the bed shaker jack to the back of clock at VIBRATOR INPUT (15).

French

MODEL:RHK009 093-BC009-052

ALARM CLOCK WITH BED SHAKER
MODEL SBC575ss

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BACK

Español

SONIC ALERT'S 1 YEAR LIMITED WARRANTY

The Sonic Boom Alarm Clock Model SBC575ss is warranted against manufacturing defects in materials and workmanship for 1 year from the date of purchase. Within this period Sonic Alert will repair or replace at our option the SBC575ss without charge for parts and labor. Simply contact our customer service department to obtain an RA (return authorization number) and instructions on how to process your product for return. Call 1-888-864-2446 or e-mail to RMA@sonicalert.info. You'll need a copy of your receipt to process the return authorization.

Sonic Alert
Made in China
Sonic Alert, Troy, MI 48083
www.SonicAlert.com 1-888-864-2446

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