

# Technical Datasheet

### RPB® T200™ Respirator

#### **DESCRIPTION**

The RPB T200 is a lightweight multipurpose respirator designed specifically for the Healthcare and Life Science industries, relieving workers of breathing difficulties and fatigue experienced with N95 use. Available with either a head harness or bump cap for head protection and multiple hood options, the T200 has the ability to adapt to different environments that need face protection and respiratory protection. Using the T200 or T-Link hoods provides an APF of greater than 1000 (CE APF 40 - TH3) with all configurations. It also gives facilities the ability to adapt the base product to suit multiple applications without having large inventories of replacement parts. The T200 has been approved by NIOSH for use with the RPB PX5® PAPR or with Supplied Air. It is CE approved with the PX5 PAPR.

This respirator does not require face fit testing.

There are four variations of respirator hood available:

- Tuchem® 2000 Face Seal Hood which allows for the wearer to easily use a stethoscope with open access to the ears.
- Tychem® 2000 Shoulder Length Face Seal Hood also allows stethoscope use without breaking the face seal but provides additional coverage down to the shoulders.
- Tychem® 2000 Full Hood with Neck Seal providing an Assigned Protection Factor of 1000.
- Tychem® 4000, Sealed Seams Full Hood with Neck Seal for higher levels of protection from chemicals and liquids.

For hearing protection, the optional Quiet-Slim™ Ear Defender earmuff system can be used.

#### Type description

Loose fitting powered air purifying and supplied air respirator.

## The T200 offers a number of optional accessory

items, including:	Part #
Quiet-Slim ear defender system	18-524
Clear-Vision Cassette Lens System for T200	17-811
Clear-Vision Cassette Lens System for T-LINK	17-817
Peel-off lens for T-LINK (pack of 50)	17-815



#### **APPROVALS & STANDARDS**







€ 2797

EN12941:1998 +A2:2008 TH3 PRSL RESPIRATORY PROTECTION

EN352-1:2002 HEARING PROTECTION

EN166:2001 CLASS 1F3 EYE & FACE PROTECTION

Not all configurations conform to this standar

Please visit rpbsafety.com for more details



HEARING PROTECTION

EYE & FACE PROTECTION



AS/NZS 1337.1:2010 BMP#714304 EYE & FACE PROTECTION

#### Respirator Assemblies

Part #

(Including PX5 PAPR, Breathing Tube, and Air Duct/Head Harness)

T200 Tychem® 2000 Face Seal Hood	17-208-22
T200 Tychem® 2000 Shoulder	17-208-32
Length Face Seal Hood	_
T-LINK Tychem® 2000 Full Length Hood	17-208-12

Additional configurations are available, including with Air Duct/Bump Cap assemblu and alternate hood tupes.





# **Technical Datasheet**

### RPB° T200™ Respirator

#### **SPECIFICATIONS**

#### Materials

ABS and Nylon	Injection molded parts
Polyurethane foam	Comfort padding on bump cap
HDPE	Head harness
PETG	Lenses
Tychem® 2000 or 4000	Hood
Cotton	Face Seal
	••••••••••••

The T200 is latex free.

#### Noise rating

	······································
When used with the 18-524	NRR: 21 / SNR: 28
Quiet-Slim ear defenders	

#### Operating temperature

14°F to 140°F / -10°C to 60°C

#### Storage temperature

14°F to 113°F / -10°C to 45°C <90%RH

#### Weight

T200 face seal hood, air duct and	0.71lbs / 321g
head harness	
T200 face seal hood, air duct and	0.90lbs / 407g

#### Assigned Protection Factor

NIOSH	APF 1000+
CE & AS/NZS	APF 40 (TH3)
	NPF 500

 $\textbf{Application ranges} \ (\text{including but not limited to the following}) \\$ 

Healthcare, pharmaceutical manufacturing, chemical, pesticide, and materials handling, painting (automotive, industrial, fire proofing), coating, concrete cutting, aggregate, chemical stripping, sanding, waste management.

#### Warranty

1 year. Conditions apply, visit <u>rpbsafety.com/healthcare/legal</u> for full details.

#### **REQUIRED COMPONENT OPTIONS**

#### **Required Components**

- T200/T-LINK respirator hood
- T200 air duct head harness/bump cap assembly
- Breathing tube
- RPB PX5 or flow control device and breathing airline

Head harness options	Part #
Head harness assembly	15-840
Bump cap assembly	15-850
Air Duct Assembly with Lens Mounts	15-860
Hood options	Part #
Tychem® 2000 Face Seal Hood	17-722
Tychem® 2000 Shoulder Length Face Seal Hood	17-732
Tychem® 2000 Full Length Hood	17-712
Tychem® 4000 Full Length Hood, Sealed Seams	17-713
Tychem® 4000 Full Length Hood, Sealed Seams, Extra Long	17-713-X

Replacement hoods: When the hood becomes either damaged or contaminated this can be easily removed and replaced. The T200 features low ongoing consumable costs and a reusable internal framework, including air duct and head harness/bump cap assembly.





# **Technical Datasheet**

### RPB® T200™ Respirator

#### **REQUIRED COMPONENT OPTIONS**

NIOSH approved flow control devices	Part #
PX5 <sup>®</sup> PAPR	03-801
C40™ Climate Control Device	03-501
Constant Flow Valve	03-101

Other NIOSH flow control device and fitting combinations available

#### NIOSH breathing airlines - standard hose

***************************************	<b></b>	
Fitting type	Hose length	Part #
RPB	25ft breathing airline	NV2028
RPB	50ft breathing airline	NV2029
RPB	100ft breathing airline	NV2027
Schrader	25ft breathing airline	04-342-25
Schrader	50ft breathing airline	04-342-50
Schrader	100ft breathing airline	04-342-100
RZ	25ft breathing airline	04-352-25-RZ
RZ	50ft breathing airline	04-352-50-RZ
RZ	100ft breathing airline	04-352-100-RZ
		••••••

Standard hoses max working length - 300ft Max working pressure - 300psi

#### NIOSH breathing airlines - recoil hose

Filting type	Hose length	Part #
RPB	15ft recoil hose	04-412-15
RPB	25ft recoil hose	04-412-25
Schrader	15ft recoil hose	04-442-15
Schrader	25ft recoil hose	04-442-25

Recoil hose max working length - 1 hose Max working pressure - 300psi

### Filtings and couplers

NIOSH	RPB Quick Connect
	Schrader Twist Connect
	RPB RZ Quick Connect

CE & AS/NZS approved flow control devices	Part #
PX5 PAPR	03-801-EUUK
PX5 PAPR	03-801-AUNZ





#### **Employers**

Read the T200 and the flow control device instruction manual and carry out the employer responsibilities.

Provide the manuals to all users.

#### **Product users**

Read the T200 and the flow control device instruction manual and follow the product user safety instructions.

Manuals are regularly updated.

#### Current version of manual and for other languages:

rpbsafety.com/healthcare/resources

#### Contact

RPB Safety - Global Headquarters 2807 Samoset Rd, Royal Oak, MI 48073, USA

T: 1-866-494-4599

E-mail: sales@rpbsafety.com rpbsafety.com/healthcare

